
Sperm VitalStain™

Symbols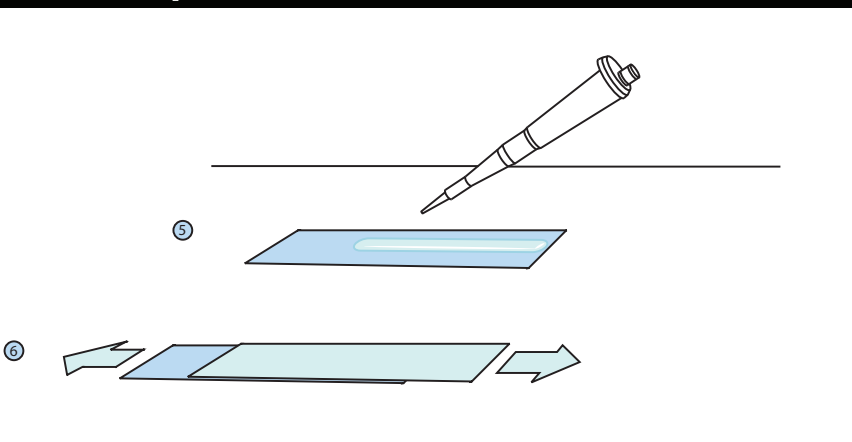

EN

Intended Use
An optimised staining technique for assessment of sperm vitality.

Components
Sodium Chloride
Eosin Y
Nigrosin
Formalin
Water

Storage and Stability
Store at 10-40°C. Under these conditions open Sperm VitalStain has a shelf-life of 24 months. The expiry date is shown on both bottles and cartons.

Precautions and Warnings

- Sperm VitalStain does not represent any kind of fire or combustion hazard. A material safety data sheet is available from the distributor or manufacturer (see nidacon.com).
- Do not use any solution which shows evidence of bacterial contamination (or if stopped accidentally comes in contact with unsterile surfaces and if it does not re-use. Reuse may result in biological contamination and/or property changes in the product.
- Rinse thoroughly with water if in contact with skin.
- If in contact with eye rinse thoroughly for 15 minutes with water and contact hospital

Reagents and Equipment

- Sperm VitalStain
- Light microscope (400-1000x magnification)
- Microscope slides
- Pipette
- Test tube
- DPX
- Immersion oil

Procedure for staining with Sperm VitalStain

- Shake the bottle of Sperm VitalStain before use
- Add an equal amount of Sperm VitalStain to the sperm sample (eg. 50µl SVS + 50µl semen). Mix well
- Incubate for 30 seconds at room temperature
- Use your conventional method to make a slide or use the method described below
- Transfer a 20µl droplet onto a labeled microscope slide with a pipette, making a stringline of fluid in the middle of the slide
- Cover this slide with a second microscope slide and, when the droplet is evenly spread between the two slides, pull them apart from each other horizontally. This method will give you two good slides
- Let the smears air dry and examine directly. Mount the slides with coverslips and DPX or equivalent mounting to store for later use
- Examine using a bright-field 40x objective or a 100x objective under oil immersion
- Count 200 sperm, the white (unstained) are classified as alive and the red or pink are classified as dead. Sperm coloured only at the neck region are assessed as alive

Symbol

- 1. Temperature limit
- 2. Use by - see label
- 3. Contains sufficient for n tests
- 4. Batch code
- 5. Consult instructions for use
- 6. CE Mark
- 7. Manufacturer
- 8. Intended Use
- 9. In vitro diagnostic medical device

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Nidacon International AB